

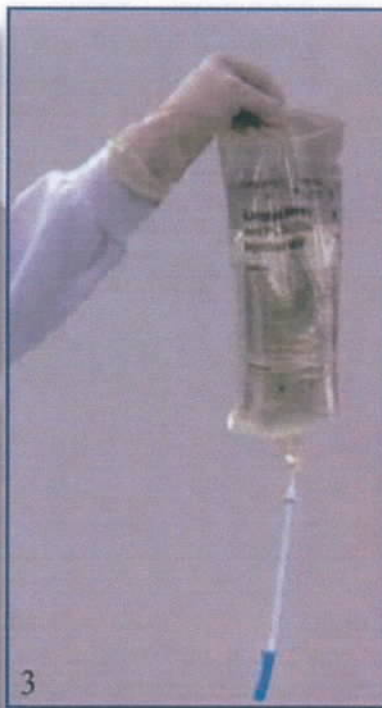


Bag Decanters

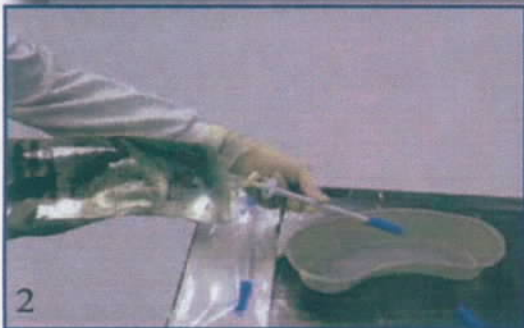

PMP's BAG DECANTER is an economical, single use product designed to facilitate the aseptic transfer of fluids from flexible IV containers. The decanter's spout can be resealed to stop the flow of the IV bag's liquid contents as needed.

*The product is easy to use in four simple steps.
(See below)*

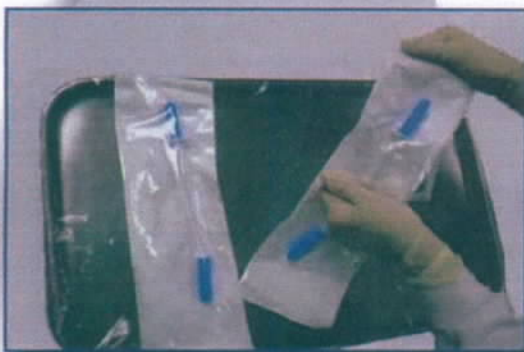

- 1.) Firmly grasp the decanter's container pouch at the end and peel open to expose the spiked end of the Bag Decanter.
- 2.) Hold the product (still in the container pouch) and remove the protective cap from the spiked end.
- 3.) Firmly insert the spike into the IV bag outlet, then remove the packaging from the spout.
- 4.) Remove the spout cap and squeeze the container to dispense the contents as needed.